



ASAHI Miracle Neo 3

PCI Guidewire



YOUR DREAMS. WOVEN TOGETHER.

ASAHI Miracle Neo

Challenge with safety



Characteristic Tip Design

Designed to maintain intraluminal position.

- Blunt tip
- 0.36mm (0.014inch) non-tapered
- Non-coated ball tip
- 1mm Pre-shape

Blunt tip
(ASAHI Miracle Neo 3)





Micro cone tip
(ASAHI Gaia Series)



Tip Flexibility

Improved smooth rigidity line.

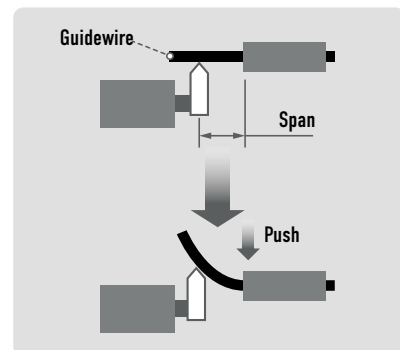
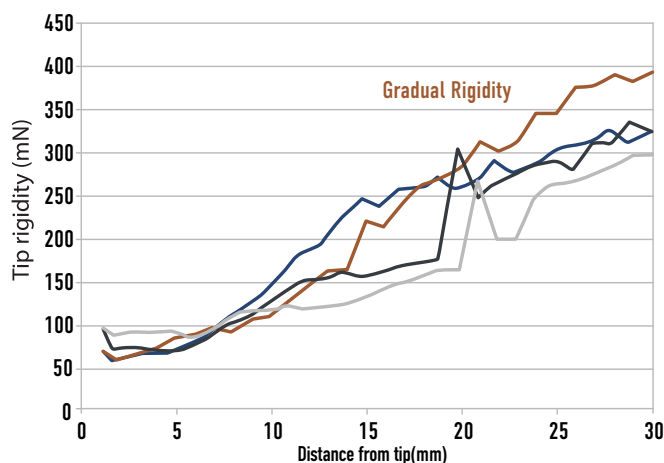


— ASAHI Gaia Second — ASAHI Miracle 3
— ASAHI Miracle Neo 3 — ULTIMATEBros 3

*The tip load values of our guide wires are nominal values assuming the force that tip of guide wire pushes lesion on the basis of company standardized test. There are individual differences in values of actual devices.



Lubricity

Better lubricity and coating durability.

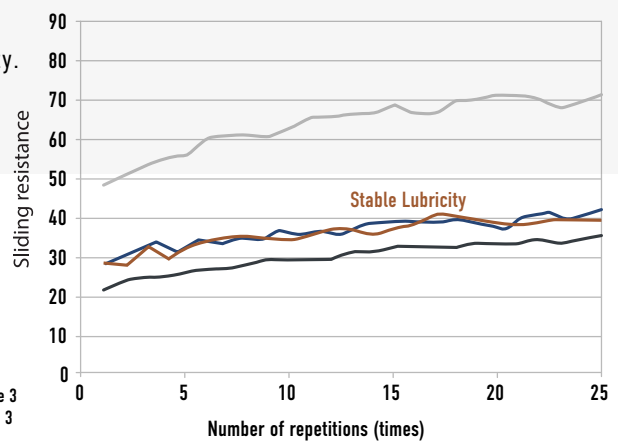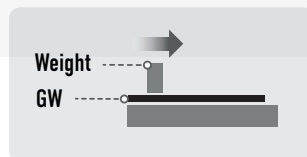

Torque Response

Superior torque response without whip motion enabling excellent maneuverability.

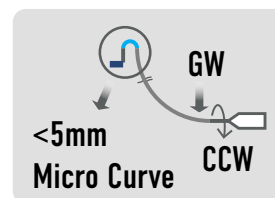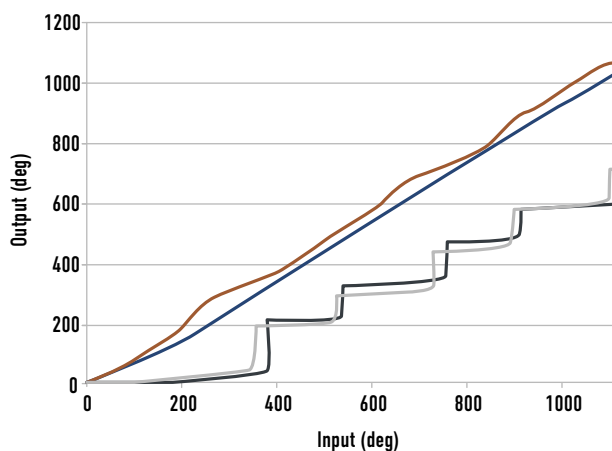


ASAHI Gaia Second ASAHI Micacle 3
ASAHI Miracle Neo 3 ULTIMATEBros 3

*All the data was obtained by company standardized test, which may differ from industry standardized tests.
*All the data does not prove that all devices have exactly the same performance with the samples used for these tests.


Structure & Ordering Information							
Product	Catalog No.	Diameter	Coating	Usable Length	Coil Length	Radiopaque Length	Tip Shape
ASAHI Miracle Neo 3	AH14R012P	0.36mm (0.014inch)	Hydrophilic Coating (SLIP-COAT®) 40cm	190cm	10.5cm	10.5cm	Pre-shape
	AH14R312P			300cm			

ASAHI Miracle Neo 3

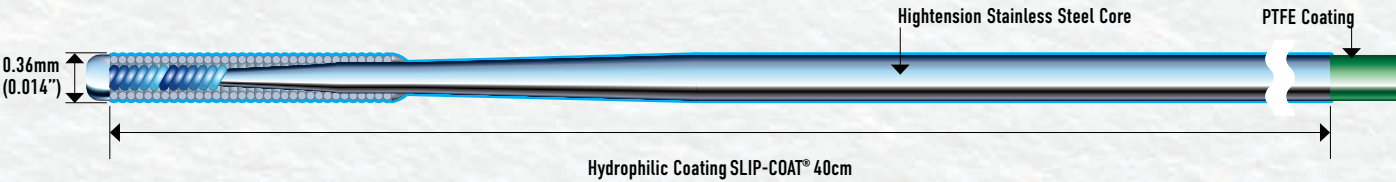

ASAHI Composite Guidewire Series with ACT ONE

ACT ONE is a multi-wire coil using our core technologies for wire drawing, wire forming, and torque. It optimizes the balance of torque force, torque response, durability, and rigidity in guidewires and microcatheters for high performance.

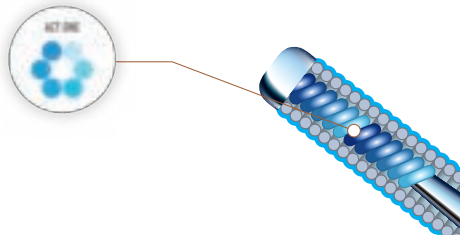

Not all the shapes and sizes are available in all markets. Please check availability with your sales representative.

©2024 ASAHI INTECC CO., LTD.
Coated with SLIP-COAT® coating.
“ASAHI Miracle” and “ACT ONE” are trademarks or registered trademarks of ASAHI INTECC CO., LTD. in Japan and other countries.
Some materials/graphics were designed & produced by Vascular Perspective Ltd.
www.vascularperspectives.com.

Manufactured by:

Contact:



ASAHI INTECC CO.,LTD.

3-100 Akatsuki-cho, Seto, Aichi 489-0071 Japan
Contact phone number: +81-561-48-5551
www.asahi-intecc.com



ASAHI INTECC EUROPE B.V.
Strawinskylaan 1461, WTC Tower 10 14th floor, 1077XX, Amsterdam, The Netherlands
Contact phone number: +31-20-794-0642
www.asahi-intecc.com